

## N95 MASK COVERS WITH ELASTIC Tutorial

Please start with 100% good quality cotton fabric, including batiks. **FABRIC SHOULD BE PRE-WASHED.** This is very important, as the covers will be laundered in a hospital setting!

**PLEASE do not include any lining materials! These may hurt those who are using these!**

### Easy Mask Cover with ties

**2 pieces of fabric, 7" by 10"**

**2 pieces of ¼" elastic, 6 inches long**

Pin your elastic to the right side of one piece of fabric, as in the picture.



Place second piece of fabric right side down on the above, and re-pin if necessary.



Sew around the piece, leaving a 3" opening in one long side. Use a ¼" seam allowance, turning at the corners as necessary.



Turn right side out. Press if necessary.



Make tick marks with chalk or pins. Start 1¼" from one long edge. Make marks every half inch seven times. Fold third line up to first, fifth to third, etc. to make three ½" pleats in each side, orienting the same way on both sides. The mask should measure about 3-3½" along the sides. Pin in place.



Topstitch around the outer edges twice, once at 1/8", and then at ¼".